



QUBÉ

THE COOL BEAUTY TOOL

BEAUTY AND PAIN RELIEF THROUGH COLD



DANGER

Store protected.
Do not use if broken or damaged,
dispose of in accordance with local
regulations. If the gel comes into
contact with skin, rinse with water
without rubbing.

WHAT IS QUBE?

It is a Class I medical device that consists
of a coolable and reusable cube.

WHAT IS QUBÉ USED FOR?

Support with aesthetic therapy and
cryotherapy

WHO IS IT SUITABLE FOR?

MEDICAL BEAUTY TREATMENTS

- Fillers and laser treatments.
- Lip treatments (tattoos, fillers,
bioregeneration).
- Biostimulation and biorevitalization.
- Aesthetic and regenerative medicine
treatments.

- Adjuvant in reducing eyelid
edema.
- Aid in reducing fat bags
on the eyelids.

BEAUTY TREATMENTS

- Epilation and depilation.
- Facial cleansing treatments

- Aesthetic tattoos.
- Improvement of rosacea.
- Adjuvant in reducing eyelid edema
- Aids to reduce the signs of fat bags
on the eyelids.

Can the cube be used over a longer period of time?

The cube can be used as directed by a doctor and as a cooling device for non-medical purposes.

When is it not recommended?

It is not indicated for the treatment of traumatized body parts when and/or complicated by ischemic lesions or vasculitis, or in areas where sensory perception is impaired due to damage

to the peripheral nervous system. The cold carrier should not be brought into direct contact with open wounds, ulcers and surgical wounds as healing can be impaired.

TYPE OF APPLICATION

Place the product in the enclosed bag and place in the freezer for at least 6 hours (the product absorbs the maximum amount of cold after 24 hours), remove from the freezer and remove the product from the bag, apply to the

affected area until the temperature of the cube corresponds to the skin or for the specified period of time.

HOW TO CLEAN

It can be cleaned with normal non-aggressive detergents and water, to disinfect use a solution of hydrogen peroxide, do not put in the oven or microwave.



SKU Q2202
LOTE 2113230568
MFD 01/2022

DISTRIBUTED by
koome Care & Cosmetics SL
Carrer Zurbaran 1
Roses (Spain)

MANUFACTURED by
IRIANS Monaco
7 Av. Saint Roman
98000 Monaco
Principality of Monaco

